Invisalign™ Stickables instructions for use. ⚠

ENGLISH

Accessory description:

Invisalign Stickables are accessories that can be used to decorate the outside of Invisalign® clear aligners. This product is available exclusively from Align Technology Inc. Patent is pending.

Indication for use

Invisalign Stickables are intended to be used in accordance with the Instructions For Use as decorative accessories only with Invisalign® clear aligners that are made from Invisalign® SmartTrack material.

Precautions:

- The use of the Invisalign Stickables has not been tested on Vivera retainers, non-SmartTrack grade aligner material, any third party clear aligners and retainers, any other orthodontic supplies such as brackets, retainers, expanders and others, any body parts, e.g. teeth, tongue, nails and skin.
- It is not recommended to apply Invisalign Stickables in areas of crowded or heavily overlapped teeth, short clinical crowns, interior of the aligner, incisal or the lingual part of the aligner, teeth with attachments, precision cuts, or precision wings, the eruption compensation and pontic area of the aligner and compliance indicators. Application of Invisalign Stickables in any of these areas may result in stickers having folds with sharp edges, stickers hiding attachments or wrapping around to the inside of the aligner and blocking the erupting tooth. It is recommended to place one sticker on the outer aligner surface covering one tooth length on the buccal or the outer area of upper anterior teeth – centrals, laterals or canines. Ensure that sticker fits on the crown surface of the aligner and that it does not hang over the incisal or gingival part of the aligner.
- Activities which could alter, damage, loosen, or dislodge the Invisalign Stickables should be avoided. Loosened Invisalign Stickables may be accidentally inhaled or swallowed by patients or could cause other damage to the patient. If stickers are swallowed, it is recommended to allow for these to pass naturally through the digestive system. If the patient is choking and coughing, stay with the person and encourage him/her to cough until the obstruction is cleared. If the patient cannot speak, or is not getting oxygen, seek emergency help immediately. Inform Invisalign Doctor if repeated loosening of the Invisalign Stickables is noticed or for any other questions or concerns. If any unusual symptoms are exhibited, consult health care practitioner (or physician) immediately.
- If a reaction to the stickers is exhibited, discontinue use and contact Invisalign Doctor.

Instructions for use

- Invisalign Stickables should be applied on the outside of aligners that are fresh out of the package. Note: It is recommended to wipe aligners with a tissue prior to applying stickers.
- Sticker adhesiveness may be also reduced in instances when aligners are handled by hands or aligners that are wet due to the presence of oil from

hands, saliva or liquid.

- If aligners are not fresh out of the package, clean and dry aligners thoroughly as per the Invisalign instructions for use before applying stickers.
- Place one sticker on the aligner outer surface covering one tooth length on the buccal area of upper anterior teeth – centrals, laterals or canines. Ensure that sticker fits on the crown surface of the aligner and that it does not hang over the incisal or gingival part of the aligner. Note: stickers can be placed on aligner areas with power ridges.
- It is recommended to replace stickers every 3 days. If stickers are beginning to peel or lift off the aligner, please refer to the sticker removal instructions to replace and reapply a new sticker.
- Invisalign clear aligners that are decorated with stickers should be worn and cleaned as per Invisalign system instructions for use.

Invisalign Stickables placement instructions (Figure 1):

- Slowly lift a sticky rectangular applicator (which has sticker underneath it) with a thumb and an index finger on one hand while holding the sheet of stickers with the other hand. Gently take off the applicator with a sticker underneath it from the sheet.
- Use applicator to line up sticker fully on the buccal or the outer part of the aligner.
- Make sure that a circle with the sticker design is right side up: the top of the design should be oriented towards the gingival part of the aligner.
- Make sure that a circle with the sticker design is lined up at the center of the crown and that there are no loose edges of the circle with the sticker design that are outside of the crown area.
- Use an index finger to press firmly and gently, rub so the sticker fully contacts the aligner.
- Slowly remove the applicator, rolling it backwards in order to remove the applicator without removing the sticker.
- Finish pressing the sticker so that it fully contacts the aligner.


Invisalign Stickables removal instructions

- Gently use a scrubbing motion with a finger to remove the sticker from an aligner. Toothbrush and water may be used to assist with a removal and cleaning process.
- Make sure that all adhesive from the sticker is off by wiping aligner surface clean with a tissue.
- Discard used sticker, clean the area thoroughly with a tissue, and apply a brand new sticker if desired.

Invisalign Stickables storage instructions

- Store at room temperature. Avoid direct sunlight.

Figure 1: How to apply Invisalign Stickables

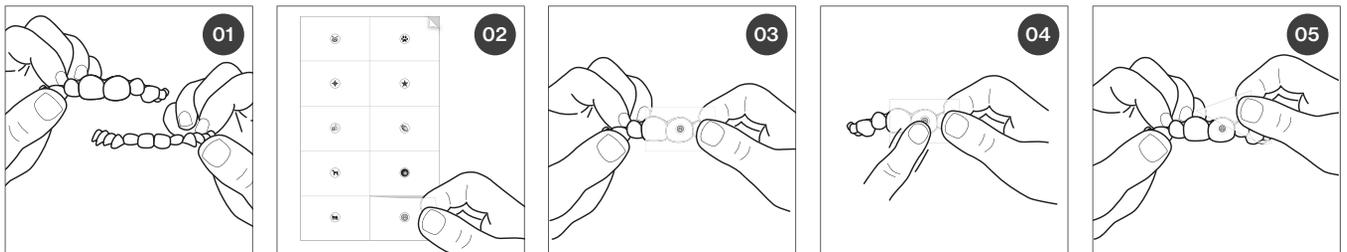Start with clean and dry aligners. Identify which are for upper / lower teeth.

Note: It is recommended to wipe aligners with a tissue prior to applying stickers.

Select the sticker to apply and peel off the clear rectangle that holds it.

Place the sticker right-side up on the upper aligner.

Rub the sticker on the aligner firmly to adhere.

Slowly peel away clear rectangle and discard, leaving the sticker on the aligner.

Note: if sticker adhesiveness is reduced during application, wipe aligner surface clean with a tissue before reattempting sticker application.

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